Navodilo za uporabo VIVAMEL – MEDICINSKI KOSTANJEV MED
Pred prvo uporabo izdelka Vivamel medicinski kostanjev med natančno preberite navodilo. Navodilo shranite. Morda ga boste želeli ponovno prebrati.

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Instructions for use VIVAMEL – MEDICAL CHESTNUT HONEY
Carefully read the instructions before using the Vivamel medical chestnut honey for the first time. Keep the instructions. You may want to read them again in the future.

WHEN TO USE THE VIVAMEL MEDICAL CHESTNUT HONEY?

Vivamel medical chestnut honey is intended for care and healing of wounds, acute and chronic, as well as surface wounds, abrasions and cuts, burns, diabetic ulcers, infected and deep wounds.

HOW DOES THE VIVAMEL MEDICAL CHESTNUT HONEY WORK?

Vivamel medical chestnut honey in tube has a high sugar content (up to 80%), low water content (up to 17%) and acidic pH. Due to its physical properties, it has a high osmotic activity, which creates an unfavourable environment for the growth of microorganisms, stimulates wound cleansing and promotes wound healing. Honey enables moist healing and forms a mechanical protection on the wound.

HOW TO USE THE VIVAMEL MEDICAL CHESTNUT HONEY?

1. First clean the wound under running tepid tap water or with physiological saline.
2. Gently dry the wound and the area around it.
3. When you open the tube for the first time, pierce the seal of the tube by inverting the cap, and press a small quantity of honey on a sterile compress and discard it. Apply the medical chestnut honey from the tube to the pad of the sterile bandage or to a sterile primary dressing of your choice; for example, alginate dressing, non-woven compress, polyurethane foam or any other appropriate dressing.
4. Place the pad of the bandage or the dressing so as to fully cover the wound. Fix the adhesive part of the bandage on the surrounding area of the skin. Secure the non-adhesive dressing with an appropriate retention bandage. In case of excessive wound drainage, place a highly absorbent dressing over the primary dressing and secure it with a retention bandage.

Initially change the dressing once a day and later every two to three days, depending on the quantity of wound drainage. Follow the procedure described above.

KONTRAINDICATION - WHEN NOT TO USE THE VIVAMEL MEDICAL CHESTNUT HONEY?

If you are allergic to honey, do not use the Vivamel medical chestnut honey.

WARNINGS

Always wash your hands before and after using the product. The Vivamel medical chestnut honey is not intended for ingestion. If ingested, it may cause blood sugar in diabetics to rise or hypersensitivity reaction in people allergic to honey. Avoid contact with eyes or mucosa. If the product comes into contact with eyes or mucosa, rinse with abundant quantities of water. Use up the Vivamel medical chestnut honey within four months after first opening the tube. After every use close the tube well in order to prevent honey from leaking out and to prevent possible ingress of air and microorganisms into the tube. The product is sterile only until it is opened for the first time. If the tube is damaged, don't use it. Honey from the tube may be applied only on a sterile dressing in order to prevent contamination of the remaining content in tube. Clinical trials of wound healing with Vivamel medical chestnut honey in diabetics have shown very good results, but treatment should certainly take place under doctor's supervision.

Any serious complication caused by a medical device should be reported to the manufacturer or the competent authority of the member state where the user resides. If you have any doubts or questions regarding the medical device and its potential risks, seek advice from a healthcare professional or pharmacist.

WHAT ARE POSSIBLE SIDE EFFECTS?

Due to a high osmotic activity of honey, which is based on removal of water molecules from the wound, a burning sensation or a mild pain may appear after applying honey to the wound. The pain usually goes away after 30 minutes to two hours maximum after the application. If the pain is unbearable, stop using the Vivamel medical chestnut honey. Notify your doctor or pharmacist about any other unwanted side effects.

HOW TO STORE THE VIVAMEL MEDICAL CHESTNUT HONEY?

Store the Vivamel medical chestnut honey in a dry place at a temperature of 5°C to 35°C in a horizontal position or with the cap upwards. Do not place the tube on the cap, because next time you use the honey it will leak out uncontrollably. Do not expose the product to direct sunlight.

REF 19705 VIVAMEL – MEDICAL CHESTNUT HONEY 20 g
REF 19715 VIVAMEL – MEDICAL CHESTNUT HONEY 20 g x 5
REF 19702 VIVAMEL – MEDICAL CHESTNUT HONEY 50 g

CE
1304

Tosama d.o.o., Vir, Šaranovičeva cesta 35, 1230 Domžale, Slovenia; www.tosama.si

TOSAMA Since 1923

Symbol legend:

Single Patient multiple use
Single sterile barrier system
Medical device
Consult instructions for use
Caution
Up to 35°C
Up to 35°C

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