Navodilo za uporabo VIVAMEL – ALGINATNA OBLOGA Z MEDICINSKIM KOSTANJNEVIM MEDOM

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Instructions for use VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY

WHAT IS VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY?

Vivamel – alginate dressing with chestnut honey is a sterile dressing in the basis of calcium alginate that is impregnated with medical chestnut honey entirely.

Medical honey Chestnut medical honey has a high sugar content (up to 80%), low water content (up to 17%) and acidic pH.

Calcium alginate Alginatne dressing is made of calcium alginate fibre, which is a natural polymer, gained from brown seaweed.

WHICH WOUNDS ARE SUITABLE FOR TREATMENT WITH VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY?

- lower leg ulcers • diabetic ulcers • pressure ulcers • infected wounds • malodorous wounds • surgical wounds • traumatic wounds • burns • moist necrosis

ADVANTAGES OF USING VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY AND ITS EFFECT:

HOW TO USE VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY?

1. First, rinse the wound with lukewarm running drinking water or saline.
2. Gently dry up the wound and its surrounding area.
3. Take the dressing with protective transparent foil out of the sterile pouch.

For easier application of Vivamel – alginate dressing with medical chestnut honey use sterile medical devices, e.g. sterile tweezers or tongs.

HOW OFTEN SHOULD ONE CHANGE VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY? At the beginning of the healing process, it is recommended that the dressing be changed every two days.

EVERY TIME BEFORE CHANGING A DRESSING, REMOVE ANY REMAINING ALGINATE OR HONEY FROM THE PREVIOUS DRESSING AND DISPOSE OF IT AS

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STORING VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY

Issue date: May 2025 Symbol legend: Single use only Expiration date Caution Medical device

STORING VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY 5x5 cm

STORING VIVAMEL – ALGINATE DRESSING WITH MEDICAL CHESTNUT HONEY 10x10 cm

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